DETAX

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INTENDED USE

Temporary luting cement for dental implant-supported restorations

INDICATION

implantlink® semi Classic

Temporary fixation of implant-supported temporary and permanent restorations.

implantlink® semi Forte

Temporary fixation of implant-supported temporary and permanent restorations on individual, reduced and small implant abutments.

implantlink® semi Xray

Temporary fixation of implant-supported temporary and permanent restorations with reinforced adhesion.

PATIENT TARGET GROUP

Persons being treated in the context of a dental procedure.

INTENDED USERS

Dentist

PRODUCT DESCRIPTION

implantlink® semi – Dual-curing, semi-permanent resin-based implant cement for occlusally stable cementation of superstructures. In the gel phase, residues can be easily removed in large pieces, without unwanted particulate residues. Thanks to the ideally adjusted adhesive properties, the bonded restorations can be detached without damage (e.g. with Crown Butler, Corona Flex), even after prolonged wear. For all material combinations, eugenol-free, odorless, and tasteless.

implantlink® semi Classic – With regular adhesion, force-fit, occlusally stable cementation for standard abutments.

implantlink® semi Forte – With increased adhesion for fixation on individualized, adhesion-area-reduced or small abutments.

implantlink® semi Xray – radiopaque, with increased adhesion and compressive strength.

PROCESSING

1. PREPARATION AND CLEANING OF THE IMPLANT ABUTMENTS AND INTERNAL SURFACES OF THE RESTORATION

Check the accuracy of fit and freedom from tension, the occlusion and contact points, before proceeding with the integration. Before cementing, degrease the abutment and superstructure, and clean (e.g. with alcoholic solutions) and dry them thoroughly (Fig. 1).

2. MIXING AND APPLICATION

The squeezing is done with the mini-mix system. Insert the plunger into the cartridge body. Remove the cartridge seal by turning it. Before fitting the mixing cannula, squeeze out a small amount of material until material is being uniformly emitted from both outlet openings (Fig. 2). Attach the mixing cannula in accordance with the guides on the cannula and cartridge, and lock it by turning it in the opposite direction (Fig. 3). Emit material with even pressure. Spread a small amount of material before use and check visually that the base and catalyst pastes are homogeneously mixed (Fig. 4). Only then can the metering be carried out individually. After use, leave the mixing cannula on the cartridge until the next application.

3. INSERTING THE RESTORATION

Apply a thin layer of implantlink® semi to the inner side of the superstructure (Fig. 5). Immediately afterwards, apply slight pressure to the implant abutments to bring them into position (Fig. 6), then press on firmly. Remove excess material after approx. 2 – 3 minutes with a suitable instrument (Fig. 7).

4. OPEN TIME / LIGHT CURING

The open time at room temperature is approx. 60 sec. Curing at body temperature takes place within 5 – 6 min. Where direct exposure is possible, the curing can be supported and shortened at any time by light exposure (marginal area, protrusions). An exposure time of 20 seconds per area is recommended for this (Fig. 8). Where translucent materials (e.g. thin layers of ceramics, temporary K&B materials) are used, it is also possible to expose through the material, extending the exposure time per area to 30 – 40 seconds.

SAFETY INFORMATION

- ▶ Only for the specified intended use by trained specialists.
- ▶ Avoid direct contact with the uncured material.
- ▶ Wear personal protective equipment (protective gloves, goggles) when handling the uncured material.
- ▶ In case of stiffness or clogging discard the cartridge; do not use force for application!
- ▶ Do not roughen or sandblast the surface of the abutment.
- ▶ When removing excess material, do not damage titanium surfaces underneath; possibly use plastic instruments.
- ▶ Do not leave any cement residues of the material in the mouth or sulcus.
- ▶ Do not use adhesion-reducing additives.
- ▶ Not suitable for temporary fixation of hybrid restorations made of natural abutment teeth and implant-supported restorations.
- ▶ Do not use adhesion-reducing additives.
- ▶ Manual withdrawal and mixing of the components is not recommended.
- ▶ The fixation should be checked after approx. 3 – 6 months. After that, integrate the restoration definitely or again temporarily.
- ▶ Wear suitable personal protective equipment (protective gloves, goggles, face mask) during the finishing of the cured material.
- ▶ Refer to the relevant safety data sheet for hazard and safety information.

NOTES

- ▶ DETAX does not accept liability for any damage caused by misuse.
- ▶ Read and understand the safety data sheet!

For users and/or patients:

Any serious incident that has occurred in connection with this product must be reported immediately to incident@detax.com and to the competent public authority of the country in which the user and/or patient resides.

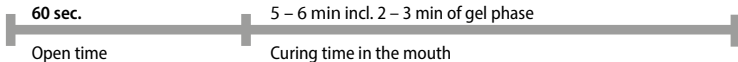
CONTRAINDICATION

Do not use this material in case of known allergies to any of the ingredients, or contact allergies. Undesirable effects are not to be expected in case of proper application. However, as a matter of principle immune reactions such as allergies or irritations cannot be ruled out. In case of doubt, we recommend performing an allergy test before application.

DISPOSAL

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.


AUTOPOLYMERIZATION:

LIGHT CURING:

Technical specifications	implantlink® semi Classic / Forte / Xray
Mixing volume	5 ml (mini-mix)
Dosage	4:1
Mixing cannula	 brown Item №: 02591
Product color Base	white-opaque
Product color Catalyst	semi-transparent
Open time *	approx. 60 sec *
Gel phase **	approx. 2 – 3 min. *
Curing time in the mouth	approx. 5 – 6 min.
Curing time in case of light curing	approx. 20 sec.
Radiopacity: Xray	≥ 100 % (aluminum)

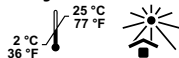Storage:

Processing: At 23 °C ± 2 °C,
50 ± 5% rel. humidity

*from start of mixing

Higher temperatures shorten, lower temperatures extend the indicated times.

**Time for removal of excess material

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implantlink® semi Classic

Standard packing

cartridge of 5 ml mini-mix 4:1
10 mixing cannulas, brown

03092

implantlink® semi Forte

Standard packing

cartridge of 5 ml mini-mix 4:1
10 mixing cannulas, brown

03371

implantlink® semi Xray

Standard packing

cartridge of 5 ml mini-mix 4:1
10 mixing cannulas, brown

02195

Mixing cannulas 4:1

brown, 25 pcs.

02591

implantlink® semi Classic

*Resin-based temporary luting cement
for implantretained restorations*

implantlink[®] semi Forte

Resin-based, temporary luting cement with increased adhesion for implant-supported restorations

implantlink[®] semi Xray

Radiopaque with enhanced adhesion, compressive strength and highest precision

DETAX

DETAX GmbH

Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany

☎ +49 7243/510-0 · 📠 +49 7243/510-100

www.detax.com · post@detax.com

Made in
Germany

06/2022

4