# FREEPRINT®

## 3D PRINTING MATERIAL

SPLINTS

# DETAX

<b>DE</b>	Gebrauchsanweisung .....	3	<b>NO</b>	Bruksanvisning .....	36
<b>EN</b>	Instructions for use .....	6	<b>PL</b>	Instrukcja stosowania .....	39
<b>FR</b>	Mode d'emploi .....	9	<b>PT</b>	Manual de instruções.....	42
<b>ES</b>	Modo de empleo .....	12	<b>RO</b>	Instrucțiuni de utilizare .....	45
<b>IT</b>	Istruzioni per l'uso .....	15	<b>SV</b>	Bruksanvisning .....	48
<b>DA</b>	Brugsanvisning.....	18	<b>SK</b>	Návod na použitie.....	51
<b>EL</b>	Οδηγίες χρήσης.....	21	<b>SL</b>	Navodila za uporabo .....	54
<b>HR</b>	Upute za uporabu .....	24	<b>CS</b>	Návod k použití .....	57
<b>LV</b>	Lietošanas instrukcija.....	27	<b>HU</b>	Használati útmutató.....	60
<b>LT</b>	Naudojimo instrukcija .....	30	<b>RU</b>	Инструкция по применению .....	63
<b>NL</b>	Gebruiksaanwijzing .....	33	<b>TR</b>	Kullanım kılavuzu .....	66

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Resin for dental 3D printing

## **INDICATION**

Splints, fixation and transfer keys

## **PATIENT TARGET GROUP**

Persons being treated in the context of a dental procedure.

## **INTENDED USERS**

Dentist, dental technician

## **SUITABLE FOR THE FOLLOWING DLP PRINTERS/CLEANING/POST-EXPOSURE**

see "Annex 1" (enclosed separately)

## **PROCESSING**

- ▶ The properties of the final product depend, among other things, on post-processing. Correct post-exposure is important for biocompatibility. Therefore it must be ensured that the light unit is in an orderly condition and that the moulds are completely cured (observe process description on page 8).
- ▶ Before use, the material in the bottle should be shaken intensively and homogenized with a bottle roller.
- ▶ Minimum material thickness for the design is 1.5 mm
- ▶ Polish surface mechanically.
- ▶ Processing temperature 23 °C ± 2 °C.

## **SAFETY INFORMATION**

- ▶ Only for the specified intended use by trained specialists.
- ▶ Avoid direct contact with the liquid material and the components before post-curing, especially in pregnant / breastfeeding women. Irritating to airways, eyes and skin (sensitization possible).
- ▶ Wear personal protective equipment (protective gloves, goggles) when handling the uncured material.
- ▶ Wear suitable personal protective equipment (protective gloves, goggles, face mask) when finishing the cured material.
- ▶ After contact with eyes rinse thoroughly with water immediately and consult a doctor.
- ▶ After contact with skin wash immediately with water and soap.
- ▶ Biocompatibility is only guaranteed with complete polymerisation.
- ▶ MMA-free.
- ▶ Refer to the relevant safety data sheet for hazard and safety information.

## **CLEANING**

**By dental laboratory / practice**

In dental laboratories and practices, the cleaning is done in ultrasonic baths.

**By the patient**

For daily cleaning, use lukewarm water, a soft toothbrush, liquid soap or detergent. Then rinse well afterwards. Cleaning tablets can be used following the manufacturer's instructions.

**Note**

- ▶ Do not use toothpaste; the abrasive particles are coarse and scratch the surface.
- ▶ Do not use any acids or solvents.

## NOTES

- ▶ DETAX shall not be held liable for any damage caused by misuse.
- ▶ Always keep container tightly sealed, immediately close the container carefully after each use.
- ▶ Please follow the instructions on the safety data sheet!

### **For users and/or patients:**

Any serious incidents occurring in relation to this product should be reported immediately to [incident@detax.com](mailto:incident@detax.com) and to the competent authority of the Member State in which the user and/or patient is established.

## STORAGE

- ▶ Store **FREEPRINT® SPLINT 2.0** in a dry place (at 15 °C – 28 °C) and away from light. Even slight exposure to light can trigger polymerization.
- ▶ To protect against contamination, cover the material in the tray with the lid or a glass plate.

## CONTRAINDICATION

Contains (meth)acrylics and phosphine oxide.

Some ingredients of **FREEPRINT® SPLINT 2.0** may cause allergic reactions in predisposed persons. In such cases refrain from using the product. **FREEPRINT® SPLINT 2.0** only insert intraorally in completely polymerised state.

## ADVERSE EFFECTS

Product may cause allergic reactions.

## DISPOSAL

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

### **Storage:**

### **Application:**

At 23 °C ± 2 °C

# Manufacturing

Data preparation and fabrication of the support structure according to the instructions of the CAD software manufacturer

## Construction process

Generation of a Print Job complying with machine and material parameters

## Post-processing

After raising the platform, a drip time of approx. 10 minutes is recommended. If possible, post-processing should commence immediately following the construction process.

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☎ +49 7243/510-0 · 📠 +49 7243/510-100

[www.detax.com](http://www.detax.com) · [post@detax.com](mailto:post@detax.com)

Caution: Federal U.S. law restricts this device to sale by or on the order of a dentist (or trained specialist personnel).

Made in  
Germany

05/2023

7

# Annex 1

## DIGITAL LIGHT PROCESSING (DLP) PRINTER, OPERATION SOFTWARE AND PARAMETER

Printer Manufacturer, Model	Light source	Light intensity	Operation Software	Parameter data set*
<b>Asiga Max</b> 	385 nm	6.1 mW/cm <sup>2</sup>	Composer 1.2.11	Detax_Freeprint splint 2.0 385_5
<b>Asiga Pico 2</b> 	385 nm	20 mW/cm <sup>2</sup>	Composer 1.2.11	Detax_Freeprint splint 2.0 385_5
<b>Asiga PRO 2</b> 	385 nm	5.7 mW/cm <sup>2</sup>	Composer 1.2.11	Detax_Freeprint splint 2.0 385_5
<b>Asiga PRO 4K</b> 	385 nm	7.0 mW/cm <sup>2</sup>	Composer 1.2.11	Detax_Freeprint splint 2.0 385_5
<b>Microlay Versus</b> 	385 nm	4.3 mW/cm <sup>2</sup>	Microform 1.0.3.7	DETAX Freeprint Splint 2.0 385 100 microns v5.3
<b>Miicraft Prime Series</b> 	385 nm	6.2 mW/cm <sup>2</sup>	Utility 6.3.0	Detax_Freeprint splint 2.0 385_100_PH
<b>Miicraft Ultra Series</b> 	385 nm	5.7 mW/cm <sup>2</sup>	Utility 6.3.0	Detax_Freeprint splint 2.0 UV_100

## DIGITAL LIGHT PROCESSING (DLP) PRINTER, OPERATION SOFTWARE AND PARAMETER

Printer Manufacturer, Model	Light source	Light intensity	Operation Software	Parameter data set*
<b>Rapidshape D20+/D30+/D40+</b> 	385 nm	2.0 mW/cm <sup>2</sup>	Netfabb 2020	Detax freeprint splint 2.0 385
<b>Way2Production SolFlex Series</b> 	385 nm	8.0 mW/cm <sup>2</sup>	Netfabb 2020	Freeprint splint 2.0 385 04-05-2021

\*The set of parameters includes all relevant material- and printer specific information

## CLEANING EQUIPMENT

Cleaning unit Manufacturer, Model	Cleaning process
<b>Rapidshape RS Wash</b>	Use the following settings: DETAX Freeprint-splint 2.0 uv Prior to post-exposure, check the openings, cavities and gap areas for residues. Then blow off with compressed air.
<b>Ultrasonic bath Bandelin Sonorex</b>	Clean the parts with isopropyl alcohol (purity $\geq 98\%$ ) for 3 minutes. Then thoroughly clean the openings, cavities and gap areas with compressed air. The main cleaning is performed in a separate vessel with fresh isopropyl alcohol (purity $\geq 98\%$ ) for 3 minutes. Prior to post-exposure, check the openings, cavities and gap areas for residues. Then blow off with compressed air.

## LIGHT CURING EQUIPMENT

Light Curing unit Manufacturer, Model	Curing process
<b>Dentalfarm Photopol</b>	2 x 3 min, progressiv + N <sub>2</sub> , turnaround components after 3 min
<b>NK Optik Otoflash G171</b>	2 x 2000 flashes under inert gas, turnaround components after 2000 flashes
<b>NK Optik Otoflash 250/500</b>	4000 flashes under inert gas @15 Hz

**DETAX**

DETAX GmbH

Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany

+49 7243/510-0 · +49 7243/510-100

www.detax.com · post@detax.com

Made in  
Germany

09/2023

3