# **FREEPRINT<sup>®</sup>**

## **3D PRINTING MATERIAL**

**INDIVIDUAL  
TRAYS**

# **DETAX**

<b>DE</b>	Gebrauchsanweisung .....	3	<b>NO</b>	Bruksanvisning .....	36
<b>EN</b>	Instructions for use .....	6	<b>PL</b>	Instrukcja stosowania .....	39
<b>FR</b>	Mode d'emploi .....	9	<b>PT</b>	Manual de instruções.....	42
<b>ES</b>	Modo de empleo.....	12	<b>RO</b>	Instrucțiuni de utilizare .....	45
<b>IT</b>	Istruzioni per l'uso.....	15	<b>SV</b>	Bruksanvisning .....	48
<b>DA</b>	Bruksanvisning.....	18	<b>SK</b>	Návod na použitie .....	51
<b>EL</b>	Οδηγίες χρήσης.....	21	<b>SL</b>	Navodila za uporabo .....	54
<b>HR</b>	Upute za uporabu .....	24	<b>CS</b>	Návod k použití .....	57
<b>LV</b>	Lietošanas instrukcija.....	27	<b>HU</b>	Használati útmutató.....	60
<b>LT</b>	Naudojimo instrukcija .....	30	<b>RU</b>	Инструкция по применению .....	63
<b>NL</b>	Gebruiksaanwijzing .....	33	<b>TR</b>	Kullanım kılavuzu .....	66

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## INDIKATION

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Composite for dental 3D printing

## **INDICATION**

Individual impression trays, functional impression trays and base plates

## **PATIENT TARGET GROUP**

Persons being treated in the context of a dental procedure.

## **INTENDED USERS**

Dentist, dental technician

## **SUITABLE FOR THE FOLLOWING DLP PRINTERS/CLEANING/POST-EXPOSURE**

see "Annex 1" (enclosed separately)

## **PROCESSING**

- ▶ The properties of the final product depend, among other things, on post-processing. Correct post-exposure is important for biocompatibility. Therefore it must be ensured that the light unit is in an orderly condition and that the moulds are completely cured (refer to the process description).
- ▶ Before use, the material in the bottle should be shaken intensively and homogenized with a bottle roller.
- ▶ Minimal material thickness of the tray 2.5 mm, in the grip area 5.0 mm  
Minimum material thickness of base plates 2.5 mm
- ▶ Do not use heat-based methods for disinfection or sterilisation. This could possibly deform the workpiece.
- ▶ Surface can be polished if necessary.
- ▶ Processing temperature 23 °C ± 2 °C

## **SAFETY INFORMATION**

- ▶ Only for the specified intended use by trained specialists.
- ▶ Avoid direct contact with the liquid material and the components before post-curing, especially in pregnant / breastfeeding women. Irritating to eyes and skin (sensitization possible).
- ▶ Be sure to use personal protective equipment (protective gloves and protective glasses) during processing.
- ▶ For finishing the cured material, suitable personal protective equipment must be used (protective gloves, goggles, face mask).
- ▶ After contact with eyes rinse thoroughly with water immediately and consult a doctor.
- ▶ After contact with skin wash immediately with water and soap.
- ▶ Biocompatibility is only guaranteed with complete polymerisation.
- ▶ Refer to the relevant safety data sheet for hazard and safety information.

## **NOTES**

- ▶ DETAX shall not be held liable for any damage caused by misuse.
- ▶ Always keep container tightly sealed, immediately close the container carefully after each use.
- ▶ Please follow the instructions on the safety data sheet!

**For users and/or patients:**

Any serious incidents occurring in relation to this product should be reported immediately to [incident@detax.com](mailto:incident@detax.com) and to the competent authority of the Member State in which the user and/or patient is established.

**STORAGE**

- ▶ **FREEPRINT® TRAY 2.0** is to be stored dry (at 15 °C - 28 °C) and protected from light. Minimal influence of light can already induce polymerisation.
- ▶ To protect against contamination, cover the material in the tray with the lid or a glass plate.

**CONTRAINDICATION**

Contains (meth)acrylics and phosphine oxide.

Some ingredients of **FREEPRINT® TRAY 2.0** may cause allergic reactions in predisposed persons. In such cases refrain from using the product.

**FREEPRINT® TRAY 2.0** only insert intraorally in completely polymerised state.

**ADVERSE EFFECTS**

Product may cause allergic reactions.

**DISPOSAL**

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

**Storage:****Processing:**

at 23 °C ± 2 °C

# Manufacturing process

Data processing and creation of the support structure according to the specifications of the CAD software manufacturer

## Building process

Creation of a print job in compliance with the machine and material parameters.

## Finishing process

After raising the platform, a dripping time of approx. 10 minutes is recommended. If possible, the finishing should take place immediately after the building process.

## Cleaning

see "Annex 1, CLEANING EQUIPMENT"

## Post-exposure

see "Annex 1, CURING LIGHT EQUIPMENT"

## Surface processing

Surface can be polished if necessary.

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
\*The set of parameters includes all relevant material- and printer specific information

## CLEANING EQUIPMENT

Cleaning unit Manufacturer, Model	Cleaning process
Rapidshape RS Wash	Use the following settings: DETAX Freprint-tray 385 Prior to post-exposure, check the openings, cavities and gap areas for residues. Then blow off with compressed air.
Ultrasonic bath Bandelin Sonorex	Clean the parts with isopropyl alcohol (purity $\geq 98\%$ ) for 3 minutes. Then thoroughly clean the openings, cavities and gap areas with compressed air. The main cleaning is performed in a separate vessel with fresh isopropyl alcohol (purity $\geq 98\%$ ) for 3 minutes. Prior to post-exposure, check the openings, cavities and gap areas for residues. Then blow off with compressed air.

## LIGHT CURING EQUIPMENT

Light Curing unit Manufacturer, Model	Curing process
Dentalfarm Photopol	2 x 3 min, progressive + N <sub>2</sub> , turnaround components after 3 min
NK Optik Otoflash G171	2 x 2000 flashes under inert gas, turnaround components after 2000 flashes
NK Optik Otoflash 250/500	4000 flashes under inert gas @15 Hz

**DETAX**

DETAX GmbH  
Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany  
☎ +49 7243/510-0 · 📠 +49 7243/510-100  
www.detax.com · post@detax.com

Made in  
Germany

09/2023  
3