blue eco / stone

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Großpackung **02985**
5 kg Base, Dose
5 kg Katalysator, Dose
2 Dosierlöffel

blue eco / stone

Putty for laboratories, silicone based, addition curing


blue eco – 86 Shore A

high consistency, easily kneadable, precise detail reproduction, stable in shape and volume

blue eco stone – 96 Shore A

medium viscosity, smooth initial consistency, high definition, dimensionally stable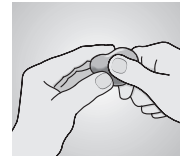


Mixing and dispensing

Remove impression material A (white) and B (blue) using measuring scoops. Mix together (mixing ratio 1:1), until a homogeneous mix or uniform colouring is achieved.

Important working hints

- Only for dental **extraoral** use by qualified personnel.
- Do not bring the components A + B into contact before mixing, do not interchange measuring scoops and jar lids.
- Do not remove material by hand, do not wear latex gloves.
- After use, close containers carefully.

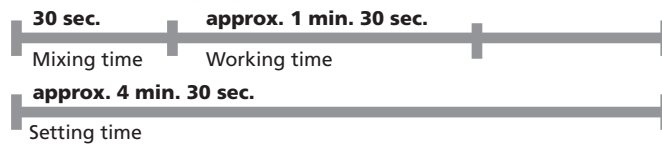DETAX

DETAX GmbH
Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany
☎ +49 7243/510-0 · 📠 +49 7243/510-100
www.detax.com · post@detax.com

Made in Germany

08/2022
4

GB

Instruction for use

Indications for use:

- Conventional indications for laboratory putties
- Fabrication of small model segments

Technical Data:

- **Mixed volume:**
blue eco: 2 x 500 ml (jars)
blue eco stone: 2 x 700 g (jars)
- **Mixing ratio:** 1:1
- **Colour code:**
base blue eco: middle blue
base blue eco stone: dark blue
catalyst: white
- **Mixing time:** approx. 30 sec.
- **Working time:**
approx. 1 min. 30 sec.*
- **Setting time:**
approx. 4 min. 30 sec.*
- **Final Hardness:**
blue eco: 86 Shore A
blue eco stone: 96 Shore A
- **Linear dimensional change:**
approx. 0.2 %
- **Application:** At 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity
- **Storage:**

* from beginning of mixing at 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity. Increased temperatures accelerate, decreased temperatures retard a. m. times.

Ordering information:

blue eco
Standard packing **02038**
base, jar of 800 ml
catalyst, jar of 800 ml
2 measuring scoops

Mega pack **02467**
base, jar of 5 kg
catalyst, jar of 5 kg
2 measuring scoops

blue eco stone
Standard packing **03117**
base, jar of 800 g
catalyst, jar of 800 g
2 measuring scoops

Eco pack **02517**
base, jar of 1400 g
catalyst, jar of 1400 g
2 measuring scoops

Mega pack **02985**
base, jar of 5 kg
catalyst, jar of 5 kg
2 measuring scoops