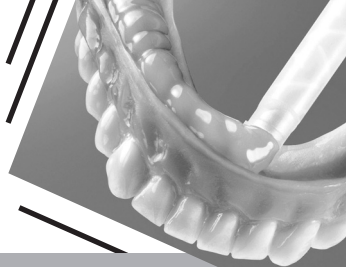
DETAX

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INTENDED USE

Definitive direct relining of dental prostheses

INDICATION

Suitable for all PMMA-based plastic materials.

Production of permanently soft relinings directly at the chairside or indirectly in the laboratory.

Removal of pressure points in the mandibular and maxillary areas.

For post-damming the A line in case of adhesion problems and flabby ridges.

As an elastic margin on maxillary dentures and for securing the position of the dentures by increased adhesion.

For elastic covering of sharp-ridged alveolar processes.

For covering implant areas and as overdentures.

PATIENT TARGET GROUP

Persons being treated in the context of a dental procedure.

Lustrol and mollosil® plus polish: Do not use for the treatment of pregnant/breastfeeding women and of children. Catalyst component contains alkyl stannosiloxanes.

INTENDED USERS

Dentist, dental technician

1. PREPARATION OF THE DENTURE

Removal of the old relining (Fig. 1) In general, an existing relining must be removed completely. Grind off the base to be relined by approx. 1 - 2 mm, depending on the situation; at the vestibular, lingual or dorsal edges, incorporate a shoulder approx. 2 - 3 mm in height with a step of at least 1 mm; round off the borders of the denture and clean the denture thoroughly.

Degreasing (Fig.2) Degrease with pure, medical alcohol and allow to dry.

Application of the mollosil® plus primer or mollosil® adhesive for adhesion (Fig. 3): All surfaces ground and to be relined to which the material is required to adhere must be evenly and completely covered with mollosil® plus primer or mollosil® adhesive. Let mollosil® plus primer mollosil® Adhesive dry for approx. 1 minute. Close the bottle immediately after use. Saliva, moisture or monomer liquids prevent adhesion of mollosil® plus or mollosil® to the denture.

2. MIXING

mollosil® plus: Application of the mixing jet / dispenser

Insert the cartridge according to special instructions (separately available, Automix1 REF 02343, Automix2 REF 02699). Remove and discard cartridge closure; do not reuse! Slightly squeeze material until it flows evenly out of both openings. Mount the mixing cannula on the cartridge and squeeze out the required material. Leave the mixing cannula on the cartridge as a closure after each use.

mollosil®: On a mixing block, mix mollosil® base and catalyst homogeneously at a ratio of 1:1 (equal strand lengths) with a spatula for 30 seconds.

3. DIRECT CHAIRSIDE RELINING

Coating of the denture (Fig. 4): Prepare the denture as described under item 1. Apply mollosil®plus cartridge material by means of a dispenser (see item 2) or mollosil® with the spatula (see item 2), taking care to avoid bubbles, to the vestibular, lingual or dorsal borders, respectively, covering the entire surface evenly. Then coat the remaining prosthetic base plate completely (layer thickness: min. 2 mm). All denture areas that come into contact with the gingiva must be coated.

Perform the functional movements (Fig. 5): Then place the denture in the patient's mouth. Let him/her close the mouth in normal occlusion. The curing process is completed after approx. 5 minutes. Then remove the relined denture from the mouth, and trim and paint it as described in item 5.

4. INDIRECT METHOD IN THE LABORATORY

Invest the denture to be relined into a flask with counter. For easier removal of the denture, cover the teeth with silicone before pouring the counter, e.g. with **Flexistone® Plus**. Remove the denture from the flask and then prepare it as described in item 1. Coat the model with a standard plaster-plastic insulation (alginate). As described under item 3, apply mixed **mollosil® plus** or **mollosil®**, respectively, taking care to avoid bubbles, evenly over the entire surface of the denture area to be relined (layer thickness: approx. 1 mm more than previously milled away). Close the flask with the counter and leave it under a press for approx. 30 min. with light pressure, or in a pressure pot at 40 - 45 °C / 104 - 113 °F in a water bath for 10 min. Then remove the denture from the flask and trim and paint it as described in item 5.

5. FINAL TRIMMING AND PAINTING

Smoothing of the borders: First remove coarse excess with a sharp scalpel or scissors (Fig. 6). Then use suitable rotating tools (e.g. **Molloplast®** cutters, pre-polishers and grinding sleeves, from approx. 8,000 RPM) with light pressure to clean and smooth the cut borders (Fig. 7).

Polish (Fig. 8): **mollosil® plus** polish or **Lustrol** (for mollosil®) should be used only to smooth the ground functional borders. Open the aluminum seal with a spatula and plug on pipettes. Pipette both liquids at a mixing ratio of 1:1 onto a mixing block, mix homogeneously with a spatula, and apply with a brush; allow to dry for approx. 5 min. at room temperature.

6. CLEANING RECOMMENDATION FOR THE PATIENT

Thoroughly brush and rinse the relined denture several times a day with a soft toothbrush and neutral soap under running water; cleaning bath: max. 10 min.

SAFETY INFORMATION

- ▶ Only for the specified intended use by trained specialists.
- ▶ Wear personal protective equipment (protective gloves, goggles) when handling the uncured material.
- ▶ Wear suitable personal protective equipment (protective gloves, goggles, face mask) when finishing the cured material.
- ▶ Adhesion is ensured only on roughened surfaces that have been coated with mollosil® plus primer or mollosil® adhesive!
- ▶ Avoid eye contact with catalyst, rinse eyes immediately with plenty of flowing water if necessary, consult ophthalmologist immediately.
- ▶ Apply mollosil® plus primer or mollosil® adhesive over the entire surface!
- ▶ All denture areas to be relined that will come into contact with the gingiva must be evenly and completely coated with mollosil® plus or mollosil®, otherwise adhesion problems may occur!
- ▶ Curing times must be complied with!
- ▶ mollosil® adhesive, mollosil® plus primer, mollosil® plus polish and Lustrol: Keep away from sources of ignition.
- ▶ Refer to the relevant safety data sheet for hazard and safety information.

NOTES

- ▶ DETAX shall not be held liable for any damage caused by misuse.
- ▶ mollosil®: Make sure that only base and catalyst components from the same batch number are used. Do not mix up tube caps!
- ▶ Read and understand the safety data sheet!

For users and/or patients:

Any serious incident that has occurred in connection with this product must be reported immediately to incident@detax.de and to the competent public authority of the country in which the user and/or patient resides.

CONTRAINDICATION

Do not use this material in case of known allergies to any of the ingredients, or contact allergies. Undesirable effects are not to be expected in case of proper application. However, as a matter of principle immune reactions such as allergies or irritations cannot be ruled out. In case of doubt, we recommend performing an allergy test before application.

DISPOSAL

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

SYMBOLS

MD Medical device

LOT see fold

**Processing:**

At 23 ± 2 °C / 73 ± 3 °F,
50 ± 5% of rel. humidity

Storage:

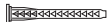
15 °C
59 °F

25 °C
77 °F

* from start of mixing at 23 ± 2 °C / 73 ± 3 °F, 50 ± 5 % of rel. humidity. Higher temperatures shorten, lower temperatures extend the indicated times.

DIN EN ISO 10139- 2 type A, soft

Technical specifications	mollosil®	mollosil® plus
Mixing volume	2 × 50 ml (tubes) 2 × 30 ml (tubes)	50-ml cartridges
Product color base/catalyst	red-brown / light blue	red-brown / white
Mixing time	approx. 30 sec manually	not applicable (Automix system)
Open time*	approx. 1 min 30 sec.	approx. 1 min 30 sec.
Curing time*	approx. 7 min.	approx. 6 min 30 sec.
Retention time in the mouth	approx. 5 min 30 sec.	approx. 5 min.
Final hardness	approx. 28 Shore A	approx. 28 Shore A
Mixing cannula	—	 Automix2, pink; item N°: 02770  Automix1; item N°: 02344

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Ordering Information

mollosil® <u>Standard packing</u> base, tube of 50 ml catalyst, tube of 50 ml adhesive, bottle of 5 ml Lustrol gloss varnish, bottles of 2 x 6 ml 2 pipettes	03005	mollosil® plus Automix1 <u>Standard packing</u> cartridge of 50 ml Primer, bottle of 5 ml Polish, bottles of 2 x 7 ml 7 mixing cannulas, 6 mm 1 grinding sleeve 1 grinding sleeve holder 2 pipettes	02436	mollosil® plus Automix2 <u>Standard packing</u> cartridge of 50 ml Primer, bottle of 5 ml Polish, bottles of 2 x 7 ml 7 mixing cannulas, pink 1 grinding sleeve 1 grinding sleeve holder 2 pipettes	02353	mollosil® adhesive bottle of 5 ml	03007	Grinding sleeves pointed, 10 pcs.	02615	mixing cannula Automix 2 rose, 48 pcs.	02707
<u>Starter kit</u> base, tube of 30 ml catalyst, tube of 30 ml adhesive, bottle of 5 ml Lustrol gloss varnish, bottles of 2 x 6 ml 2 pipettes	02274	<u>Refill packing Automix1</u> cartridge of 50 ml	02439	<u>Refill packing Automix2</u> cartridge of 50 ml	02354	Lustrol gloss varnish bottles of 2 x 6 ml	03008	Grinding sleeve holder pointed, 1 pcs.	02616	mixing cannula Automix 1 6 mm, 48 pcs.	02344
						Primer bottle of 5 ml	02440	good morning spray <u>Standard packing</u> spray bottle of 100 ml	02284		
						Polish bottles of 2 x 7 ml	02441				

DETAX

GmbH & Co. KG

Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany

Telefon: 072 43/510-0 · Fax: 072 43/510-100

www.detax.de · post@detax.de

Made in
Germany

07/2020

3

Caution: Federal U.S. law restricts this device to sale by or on the order of a dentist (or trained specialist personnel).