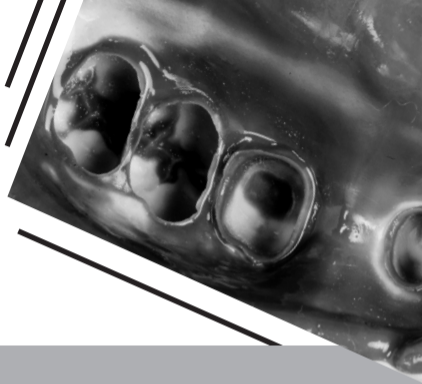
DETAX

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VPS precision impression silicone for dental prosthetics

INDICATION

onetime perfect putty – tray material for the double-mix and sandwich technique (single-step) and correction impression (two-step).

onetime perfect heavy – tray material for the double-mix and sandwich technique (single-step).

onetime perfect wash – correction material, sprayable, for the double-mix and sandwich technique (single-step) and correction impressions (two-step).

PATIENT TARGET GROUP

Persons being treated in the context of a dental procedure.

INTENDED USERS

Dentist

PRODUCT DESCRIPTION

onetime perfect putty, heavy and wash are silicone-based, addition-curing elastomeric precision impression materials.

onetime perfect materials have differing consistencies according to DIN EN ISO 4823 type 0 – 3 (see table page 11).

METERING AND MIXING IS CARRIED OUT USING DIFFERENT SYSTEMS

putty 1:1 – manual metering and kneading

Jumbo cartridge 5:1 – commonly used dynamic mixers

Cartridges 1:1 – Automix2

PROCEDURE

TRAY, ADHESIVE

Select a suitable impression tray according to the impression technique. Apply trayloc® A to optimize silicone adhesion.

MANUAL METERING AND MIXING – 1:1 ONETIME PERFECT PUTTY

The metering is carried out at a mixing ratio of 1:1 with the enclosed metering spoons. Mixing is carried out manually by kneading until a homogeneous mixture or uniform color, respectively, is reached. Make sure that only base and catalyst components from the same batch number are used. Immediately close again after use. Do not swap lids, closures or metering spoons.

DYNAMIC MIXERS – 5:1 JUMBO CARTRIDGE – ONETIME PERFECT HEAVY

Unlock and pull off the tamper-proof closure on the cartridge by lifting the locking lever (Fig. 1). Insert the cartridge into the metering unit. Start the device and spread the material until it is being evenly emitted. Discard the first inhomogeneous mass, then attach the dynamic mixer (yellow). Fixate the mixer by attaching and turning the yellow rotating ring to the right (Fig. 2). Discard the first material that comes out, approx. 3 cm. Do not fill the tray before the mixture is homogeneous. Leave the filled mixer as a closure on the cartridge until the next application. Before the next application, remove the used mixer and check the outlet openings of the cartridge for clogging; then attach a new mixer. In order to ensure reliable application, we recommend using exclusively the matching dynamic mixers by DETAX. Please read and understand the notes by the manufacturer of the mixer.

AUTOMIX2 CARTRIDGE 1:1 – ONETIME PERFECT WASH

The cartridges are emptied with the Automix2 system pistol (Fig. 3). After inserting the cartridge into the mixing gun, remove the cartridge closure by turning it. Squeeze out a small amount of material until silicone is being uniformly emitted from both outlet openings (Fig. 4). Attach the mixing cannula in accordance with the guides on the cannula and cartridge, and lock it by turning it in the opposite direction (Fig. 5). Emit material with even pressure. Spread a small amount of material before use and visually check that the base and catalyst pastes are homogeneously mixed (Fig. 6). Only then can the metering be carried out individually. After use, leave the mixing cannula on the cartridge until the next application.

DISINFECTION

After taking the impression out of the mouth, rinse it under running lukewarm water. Subsequent disinfection can be carried out with suitable commercially available disinfecting solutions according to the manufacturer's instructions.

PRODUCING THE LABORATORY MODEL

With dental plaster type III – V, this can be done already about 30 minutes after removal from the mouth.

TRAY CLEANING

Remove cured material mechanically. Placing in acetone or isopropanol dissolves trayloc® A. Use only in well-ventilated areas. Then clean and disinfect the tray as usual.

SAFETY INFORMATION

- ▶ Only for the specified intended use by trained specialists.
- ▶ In impression situations with heavily submerged areas and widely open interdental spaces, the usual block-out measures must be carried out. In the case of one-step impressions, these can be filled up e.g. with Detaseal® antilock (do not use in the preparation area, only for blocking out).
- ▶ Solutions used prior to impression taking (e.g. retraction solutions) may affect the curing reaction and must therefore be removed completely.
- ▶ Latex gloves and latex-contaminated surfaces can affect curing.
- ▶ In case of stiffness or clogging discard the cartridge; do not use force for application!
- ▶ Do not leave any residues of the impression material in the patient's mouth or sulcus, and make sure to avoid inadvertent pressing of material into areas outside the oral cavity through mouth-antrum connections or malformations of the palate.
- ▶ Cured impression materials are chemically resistant – avoid stains on clothing.
- ▶ Wear suitable personal protective equipment during the finishing of the cured material.
- ▶ Refer to the relevant safety data sheet for hazard and safety information.

NOTES

- ▶ DETAX shall not be held liable for any damage caused by misuse.
- ▶ Read and understand the safety data sheet!

For users and/or patients:

Any serious incidents occurring in relation to this product should be reported immediately to incident@detax.de and to the competent authority of the Member State in which the user and/or patient is established.

CONTRAINDICATION


Do not use this material in case of known allergies to any of the ingredients, or contact allergies. Silicone impression materials have been tried and tested millions of times; undesired effects are not to be expected in case of proper application. However, as a matter of principle immune reactions such as allergies or irritations cannot be ruled out. In case of doubt, we recommend performing an allergy test before application.

DISPOSAL

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

Technical Specifications	onetime perfect putty	onetime perfect heavy	onetime perfect wash
DIN EN ISO 4823 TYPE	type 0, putty	type 1, heavy-bodied	type 3, light-bodied
Mixing volume	2 × 250 ml can	380-ml cartridge	50-ml cartridge
Metering	1:1	5:1	1:1
Product color Base/Catalyst	turquoise / white	light green / white	purple / lilac
Mixing time	approx. 30 sec.	n/a	n/a
Open time* Min. approx.	1:30	1:30	1:30
Indwelling time in mouth min. approx.	2:00	2:00	2:00
Curing time* min. approx.	3:30	3:30	3:30
Mixer	n/a	dynamic mixer, yellow	mixing cannula pink, T-mixer pink

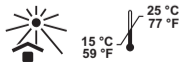Storage:

Processing: At 23 °C ± 2 °C,
50 ± 5% rel. humidity

Symbols

MD Medical Device

* from start of mixing at 23 °C ± 2 °C, 50 ± 5 %
rel. humidity. Higher temperatures shorten, lower
temperatures extend the indicated times.

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onetime *perfect*

onetime perfect putty
Standard-Pack 03288

2 jars, ea. 250 ml
2 measuring scoops

Eco-Pack 02155

8 jars, ea. 250 ml
4 measuring scoops

onetime perfect wash

Standard-Pack

03289

2 cartridges, ea. 50 ml
12 mixing cannulas, pink
12 intra-oral-tips, white

Eco-Pack

02152

8 cartridges, ea. 50 ml
48 mixing cannulas, pink
48 intra-oral-tips, white

onetime perfect heavy

Jumbo-Pack





03102

maxi cartridge of 380 ml,
base + catalyst
6 dynamic mixers, yellow
1 rotating ring, yellow

Ordering Information

onetime perfect putty <u>Standard packing</u> 03288 1 jar base each 250 ml 1 jar catalyst each 250 ml 2 measuring scoops	
<u>Eco packing</u> 02155 4 jar base each 250 ml 4 jar catalyst each 250 ml 4 measuring scoops	
onetime perfect heavy <u>Jumbo packing</u> 03102 1 cartridge base + catalyst 380 ml 6 dynamic mixers, yellow 1 rotating ring, yellow	
onetime perfect wash <u>Standard packing</u> 03289 2 cartridges each 50 ml 12 mixing cannulas, rose 12 Intra-Oral-Tips, white	
<u>Eco packing</u> 02152 8 cartridges each 50 ml 48 mixing cannulas, rose 48 Intra-Oral-Tips, white	
Dynamic mixers yellow 40 pcs. 02833	
Rotating ring yellow 1 pc. 02836	
Mixing gun 1:1 / 2:1 1 pcs. 02699	
Mixing cannulas rose 48 pcs. 02707	
T-Mixer rose 25 pcs. 03614	
trayloc® A bottle with brush, 17 ml 03098 vaporizer of, 15 ml 03392	

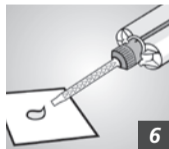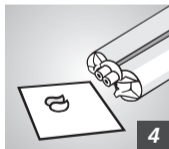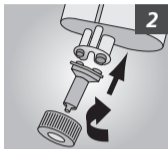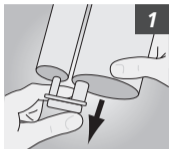DETAX

GmbH & Co. KG

Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany
Telefon: 07243/510-0 · Fax: 07243/510-100
www.detax.de · post@detax.de

Made in
Germany

11/2020
3

Caution: Federal U.S. law restricts this device to sale by or on the order of a dentist (or trained specialist personnel).